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EN English

Introduction

Thank you for purchasing Rossmax bubble air mattress. With proper care and use, your air mattress will provide you with many years of reliable treatments. We encourage you to thoroughly read this guidebook to learn about setting up and the features of your air mattress. The mattress relieves pressure by sequentially deflating and inflating alternate air cells on approximately 6 minutes timed intervals. It is intended to help and reduce the incidence of pressure ulcers while optimizing patient comfort, long term care of patients suffering from pressure ulcers.

Attention: Consult the accompanying documents. Please read this manual carefully before use.

Please be sure to keep this manual.

Product caution

1. Do not smoke on or near the pump.
2. Keep the pump away from heated surfaces.
3. Explosion risk if used in the presence of flammable anesthetics.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, please contact Rossmax service if necessary.
5. Replace fuse as marked: T1A 250V.
6. Do not use with patients who have a spinal cord injury.
7. Close supervision is necessary when this product is used on or near children.
8. Use this product only for its intended use as described in this manual. Do not use other mattress not recommended by manufacturer.
9. Do not modify this product without authorization of the manufacturer.
10. If you suspect that you may have had or are having an allergic reaction, please consult your physician immediately.

How to use

1. Place the mattress on the bed frame with the hose end at the foot section of the bed frame. Secure the mattress firmly by fixing the flags to the bed frame if available.
2. Using the integrated hangers, securely hang the pump on the bed end at the foot end or place on a soft flat surface.
3. Connect the air hoses from the mattress to the pump unit.
- Note:** Check and ensure the air hoses are not kinked or tucked under mattress.
4. Plug the pump into a wall electrical outlet. Be sure the power cord is safely away from possible hazards.
- Note:** After installation, the extra length of power cord, if any, should be neatly arranged to avoid any tripping accidents.
5. Turn on the power switch on the control panel of the pump. The pump will begin to inflate the mattress.
- Note:** The pump should only be used with mattress recommended by manufacturer. Do not use it for any other purpose.
6. While inflation, adjust the pressure setting based upon patient comfort level by turning the pressure adjustment knob clockwise to increase firmness.
- Note:** Every time the mattress is set up for use, it is recommended that the pressure first to be set to max, and then adjust air mattress weight levels to the desired softness after set up has been completed.

Clean and disinfection

1. Wipe the pump unit with a damp cloth and mild detergent. If other detergent is used, choose one that will have no chemical effects on the surface of the plastic case of the pump unit.
2. Wipe down the mattress with warm water containing a mild detergent and air dried thoroughly before use.
- Note:**
 - Do not use phenolic based product for cleaning.
 - After cleaning, dry the mattress without direct exposure sunlight.

Troubleshooting

Symptoms	Check point/Correction
Power is not ON	• Check if the plug is connected to mains. • Check for a blown fuse.
Patient is bottoming out	• Pressure setting might be inadequate for the patient, adjust higher pressure and wait for a few minutes for best comfort.
Mattress form is loose	Check if the mattress is fixed to the bed frame by flags.
No air produced from some air outlets of the air tube connector	This is normal since there is alternating mode, air outlets take turns to produce air during their cycle time.

Note: If the pressure level is consistently low, check for any leakage (tubes or air hoses). If necessary, replace any damaged tube or hoses or contact Rossmax service if necessary.

Specification

Pump unit	Dimension (L x W x H)	26.5 x 13.5 x 10 cm
	Weight	3 lbs (1.4 kg)
AC power source	AC 230V/50Hz or AC 220V/60Hz or AC 110V/60Hz	
Cycle time	6 mins	
Mattress	Dimension (L x W x H)	79" x 35" x 3" (200 x 90 x 7 cm)
Extension flags	20"(50 cm) of both sides	
Weight	5 lbs (2.3 kg)	
Weight Capacity	220 lbs (100 kg)	
Operating Condition	10°C to 40°C (50°F to 104°F), 10% to 75% RH; 700~1060hPa	
Storage & Transportation Condition	-18°C to 43°C(0°F to 110°F), 10% to 95% RH	
	Type BF applied parts	
IP Classification	IP21: Protection against harmful ingress of water and particulate matter	

EMC guidance and manufacturer's declaration

Guidance and manufacturer's declaration-electromagnetic emissions
The AM30 is intended for use in the electromagnetic environment specified below. The customer or the user of the AM30 should ensure that it is used in such an environment.

Emission test Group 1
The AM30 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emissions CISPR 11 Class B
The AM30 is suitable for use in all establishments, including domestic establishments and those directly connected to the public supply power network that supplies buildings used for domestic purposes.

Harmonic emissions IEC 61000-3-2
Voltage fluctuations/flicker emissions IEC 61000-3-3
Compliance

Guidance and manufacturer's declaration-electromagnetic immunity
The AM30 is intended for use in the electromagnetic environment specified below. The customer or the user of the AM30 should ensure that it is used in such an environment.

Immunity test IEC 60601 test level
Compliance level

Electromagnetic environment-guidance

The AM30 is intended for use in the electromagnetic environment specified below. The customer or the user of the AM30 should ensure that it is used in such an environment.

Electro